




# OWNER'S MANUAL


## Walker's Digital Hybrid EX

- 8 Channel Digital Circuitry
- 16 Band Graphic Equalizer
- 4 Programs
- Open/Closed Fit

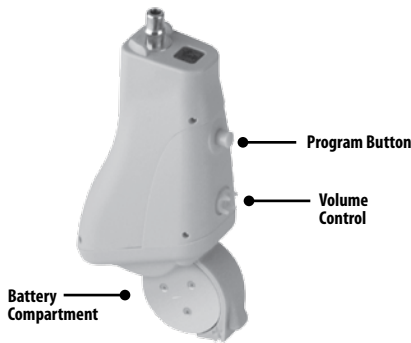


### Selecting the Right Size Tube

The Hybrid EX includes 3 different size tubes: Small, Medium and Large. Place the Hybrid EX over your ear, with the medium tube attached. If the tube is too short, the Hybrid EX's back end will be up and the front end will be down. In this case, see Step 4 to change to the large, 3B tube. If the tube is too long, the Hybrid EX's back end will be too low and the front end will be too high. In this case, see Step 4 to change to the small 1B tube. Right tubes are labeled in red print, left tubes in blue.

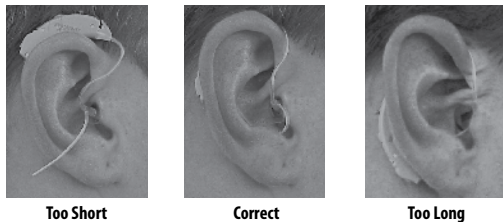

### Changing Tubes

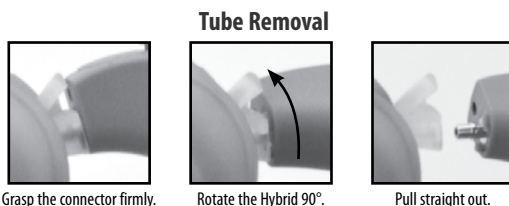


**Note:** It is important NOT to pull the tubing itself, as it can become stretched and distorted.

### Tube Attachment



To attach, align the tube assembly as shown (tube facing down)

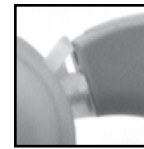


Apply gentle pressure until the tube snaps into place.

**Note:** You will feel a firm de-dent as it locks into place.

### Choosing the Right Tip

**Attention:** In loud environments where hearing protection is necessary, use the foam plug. An additional foam plug is included for the other ear. For environments that do not have loud sounds, you can use either the foam tip or the tulip tip.

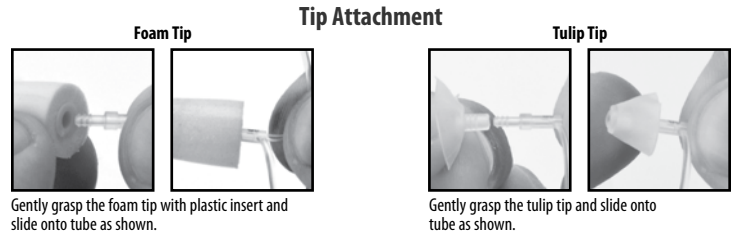

Gently grasp the foam tip with plastic insert and slide onto tube as shown.

Gently grasp the tulip tip and slide onto tube as shown.

### Battery Insertion

The Hybrid EX uses a standard size #312 battery.



- To insert, remove adhesive sticker. Any moisture on the battery surface should be wiped off prior to insertion. Open the battery door and insert the battery.
- To remove the battery, open the battery door and lift the battery out of the battery door.

**Low Battery Indicator:** When your battery is low you will hear five beeps indicating that it is time to change your battery.


**Helpful Hint:** It is recommended that you leave your battery door open when not using your instruments. This will prolong the life of your batteries.

**Caution:** Keep batteries out of reach of children or anyone who may accidentally swallow them. Never put batteries in your mouth for any reason. Do not store batteries in medication containers or where they may possibly be mistaken for medication and accidentally swallowed. Never attempt to recharge non-rechargeable batteries.

### How To Wear The Hybrid EX

**With Foam Plug:** Squeeze and roll the foam plug between your fingers. Insert into ear canal. The foam plug will gradually expand after insertion. Slide the Hybrid EX over your ear. Make sure the foam tip does not loosen. Tuck the retention line in the bowl of your ear and adjust/bend it until comfortable and secure.

**With Tulip Tip:** Grasp the Hybrid EX between your thumb and forefinger. Place it over your ear as shown. Gently push the tulip into the ear canal. Tuck the retention line into the bowl of your ear and adjust/bend it until comfortable and secure.

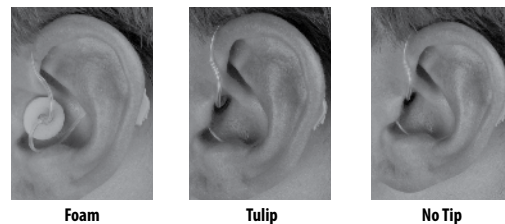

### On/Off and Volume Adjustment

**On & Off:** The Hybrid EX is turned on and off by opening the battery compartment.

**Volume Adjustment:** Adjust the volume level of the Hybrid EX by rotating the volume control. When adjusting the volume control, you will hear a single beep when the volume level is at mid-point.

## Program Selection

The Hybrid EX has 4 programs:

Program	Ideal For
1. General	Everyday Use
2. Crowds	Restaurants/Sporting Events/Large Meetings
3. Music/Theater	Concerts/Cinema
4. Nature	Outside/Birdwatching/Hunting

When first turned on, the Hybrid EX defaults to #1, the General program. To change between programs, push the program button. You will hear a series of beeps as you change programs: one beep: program # 1, two beeps: program # 2, etc.

## Caring for Your Hybrid EX

The Hybrid EX is a delicate, precision electronic device and as such, requires the special care and handling that would be afforded to any fine piece of equipment. Please read and follow the listed precautions to insure quality performance:

- Remove your batteries when your device will not be in use for prolonged periods of time.
- Do not throw or drop your device.
- Do not immerse device in liquid or water.
- Do not wear device during bathing, showering, or swimming.
- Do not expose device to excessive heat, such as a stove top, microwave, dash board of a hot car, or a hair dryer.
- Do not expose the device to chemicals, such as hair spray, insect repellent, perfume, or cosmetics.

## Common Problems & Solutions

Problem	Possible Cause	Solution
Feedback or Whistling	Ear wax accumulation in ear canal or tip Tip partially inserted	Clean tulip tip, replace foam plug Reinsert your device
No Sound	Microphone opening, tip, or tube clogged with wax or debris Battery dead	Clean tip and tubing if necessary Replace battery
Fades and/ or Intermittent	Tube and/or clip improperly attached Dirty, corroded, weak or dead battery	Reinsert device Replace battery

### LIMITED WARRANTY

Walker's warrants this product to the original purchaser to be free from defects in materials and workmanship, under normal use and conditions for a period of two years parts/90 days labor from the date of the original purchase.

Under this warranty, this unit will be replaced or repaired at our option, without charge for parts or labor, excluding batteries, for a period of 90 days after the purchase. After 90 days, the cost of labor with Shipping/Handling and Insurance of \$99.95 is the customer's responsibility.

#### This warranty is void if any of the following occur:

1. The warranty card was not sent in at the time of purchase
2. The unit is not owned by the original purchaser at the time of the warranty claim
3. The unit has been tampered with, opened or punctured
4. The unit has been immersed in water or other liquids, allowed to be filled with dirt or dust, or otherwise physically abused
5. The 24-month limited warranty has expired

This shall be the exclusive written warranty of the original purchaser and neither this warranty or any other warranty, expressed or implied, shall extend beyond the 24-month period listed above. In no event shall Walker's Game Ear, Inc. be liable for consequential damages, consequential property damage, or consequential personal injury. (Some states do not allow limitations on implied warranties or exclusions of consequential damages, so that these restrictions may not apply to you). This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

Record serial # for future reference: Serial # \_\_\_\_\_

Return bottom portion to: Walker's, Inc. • PO Box 535189 • Grand Prairie, TX 75053-5189  
Or you can register your product online at: [www.WalkersGameEar.com](http://www.WalkersGameEar.com)

### Warranty Registration

Model: Digital Hybrid EX \_\_\_\_\_ Date of Purchase: \_\_\_\_\_

Purchased From: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

Primary Use: \_\_\_\_\_

Your Name: \_\_\_\_\_

Address: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

Serial #: \_\_\_\_\_ Email: \_\_\_\_\_

Toll-free customer support 877-269-8490  
[www.walkersgameear.com](http://www.walkersgameear.com)