



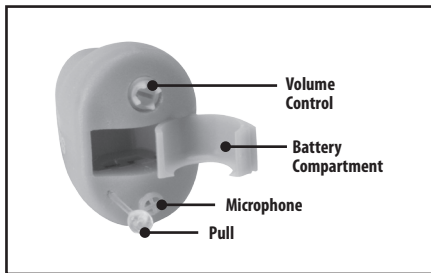

OWNER'S MANUAL

Walker's Digital Hybrid Duo

- 6 Channel Digital Circuitry
- 4 Programs
- 16 Band Graphic Equalizer
- Volume Control

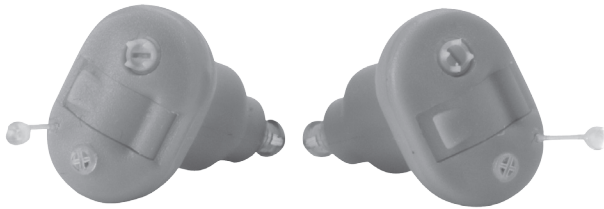

Battery Insertion

The Hybrid Duo uses a standard size #10A battery.



- To insert, remove adhesive sticker. Any moisture on the battery surface should be wiped off prior to insertion. Open the battery door and insert the battery.
- To remove the battery, open the battery door and lift the battery out of the battery door.

Low Battery Indicator: When your battery is low you will hear five beeps indicating that it is time to change your battery.

Caution: Keep batteries out of reach of children or anyone who may accidentally swallow them. Never put batteries in your mouth for any reason. Do not store batteries in medication containers or where they may possibly be mistaken for medication and accidentally swallowed. Never attempt to recharge non-rechargeable batteries.

Choosing the Right Tip

Your Hybrid Duo can be worn three ways:

Tip	Situation
No Tip	Quiet environments only
Vented 'Star' Shaped Foam Tip	Everyday environments where hearing protection is not required
Solid 'Circular' Shaped Foam Tip	Any environment where hearing protection is required

IMPORTANT: For hearing protection from sudden or sustained loud noises, the solid plug must be worn.

Each foam plug comes in three different sizes: Small, Medium and Large. Choosing the correct size is a matter of personal fit and comfort.

Replacement foam tips are available from Walker's: 877-269-8490.

Attaching the Tip to the Hybrid Duo

- Select which size tip, Small, Medium or Large
- Push the Hybrid Duo into the center of the of the foam tip, push with gentle force so the snap-lock mechanism securely engages
- Slowly pull the Hybrid Duo up, with the foam tip attached



Precautions:

- Do not insert in your ear unless the foam tip is firmly attached to the Hybrid Duo
- Stop using the Hybrid Duo if ear canal becomes uncomfortably irritated
- Do not use with a draining ear

Removing the Tip from the Hybrid Duo



- Place the Hybrid Duo in the notch on the foam tip package
- Gently pull back and the tip will come off
- Discard used foam tip - do not try to reattach a tip that has been removed
- Keep foam tips out of reach of children

On/Off and Volume Adjustment

On & Off: The Hybrid Duo is turned on and off by either opening/closing the battery compartment or by turning the volume control past the lowest setting.

Volume Adjustment: Adjust the volume level of the Hybrid Duo by rotating the volume control up for louder volume and down for softer volume. When adjusting the volume control, you will hear a single beep when the volume level is at mid-point.

Program Selection

The Hybrid Duo has 4 programs:

Program

1. General
2. Crowds
3. Music/Theater
4. Nature

Ideal For

- Everyday Use
- Restaurants/Sporting Events/Large Meetings
- Concerts/Cinema
- Outside/Birdwatching/Hunting



When first turned on, the Hybrid Duo defaults to #1, the General program. To change between programs, hold the program wand next to your ear. You will hear a series of beeps as you change programs: one beep: program # 1, two beeps: program # 2, etc.

Caring for Your Hybrid Duo

The Hybrid Duo is a delicate, precision electronic device and as such, requires the special care and handling that would be afforded to any fine piece of equipment. Please read and follow the listed precautions to insure quality performance:

- Remove your batteries when your device will not be in use for prolonged periods of time.
- Do not throw or drop your device.
- Do not immerse device in liquid or water.
- Do not wear device during bathing, showering, or swimming.
- Do not expose device to excessive heat, such as a stove top, microwave, dash board of a hot car, or a hair dryer.
- Do not expose the device to chemicals, such as hair spray, insect repellent, perfume, or cosmetics.

Common Problems & Solutions

Problem	Possible Cause	Solution
Feedback or Whistling	Ear wax accumulation in ear canal or tip Tip partially inserted	Replace foam plug Reinsert your device
No Sound	Microphone opening, tip, or tube clogged with wax or debris Battery dead Tube and/or clip improperly attached	Clean tip if necessary Replace battery Reinsert device
Fades and/ or Intermittent	Dirty, corroded, weak or dead battery	Replace battery

LIMITED WARRANTY

Walker's warrants this product to the original purchaser to be free from defects in materials and workmanship, under normal use and conditions for a period of two years parts/90 days labor from the date of the original purchase.

Under this warranty, this unit will be replaced or repaired at our option, without charge for parts or labor, excluding batteries, for a period of 90 days after the purchase. After 90 days, the cost of labor with Shipping/Handling and Insurance of \$99.95 is the customer's responsibility.

This warranty is void if any of the following occur:

1. The warranty card was not sent in at the time of purchase
2. The unit is not owned by the original purchaser at the time of the warranty claim
3. The unit has been tampered with, opened or punctured
4. The unit has been immersed in water or other liquids, allowed to be filled with dirt or dust, or otherwise physically abused
5. The 24-month limited warranty has expired

This shall be the exclusive written warranty of the original purchaser and neither this warranty or any other warranty, expressed or implied, shall extend beyond the 24-month period listed above. In no event shall Walker's Game Ear, Inc. be liable for consequential damages, consequential property damage, or consequential personal injury. (Some states do not allow limitations on implied warranties or exclusions of consequential damages, so that these restrictions may not apply to you). This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

Record serial # for future reference: Serial # _____

Return bottom portion to: Walker's, Inc. • PO Box 535189 • Grand Prairie, TX 75053-5189
Or you can register your product online at: www.WalkersGameEar.com

Warranty Registration

Model: Digital Hybrid Duo _____ Date of Purchase: _____

Purchased From: _____

City: _____ State: _____ Zip: _____

Primary Use: _____

Your Name: _____

Address: _____

City: _____ State: _____ Zip: _____

Serial #: _____ Email: _____

Toll-free customer support 877-269-8490
www.walkersgameear.com